



Canvys a division of Richardson Electronics, Ltd.

www.canvys.com

Quick Install Guide

Matrox Aurora with Image Systems FP2080 and FPP2080 displays

For additional information please refer to the [Matrox User Guide](#)

Note: Before starting driver installation connect all displays to the graphic card. A navigational VGA display connected with an **analog** cable is required for installation. This display can be disabled after setup if desired

- ☐ 1. Start computer and cancel Windows new hardware wizard
- ☐ 2. Load Canvys "M1" cd
- ☐ 3. From the menu choose **Install Matrox Aurora Driver** and choose **Yes** or **Run** on any subsequent security prompts; Install Shield will start > Follow on screen directions. Click **Next, Next, Next, Next**, and click **Finish** to restart computer.

Options:

Changing display orientation

- ☐ 1. Right-click mouse over Matrox Icon in lower right tool tray.
- ☐ 2. Select **Multi-Display setup...**
- ☐ 3. Change Display Rotation as required click **Apply**, click **Yes**.

Connecting only one grayscale display

- ☐ 1. Right-click mouse over Desktop
- ☐ 2. Click **Properties**, click **Settings** tab
- ☐ 3. Select the monitor associated with the second head of the Matrox card
- ☐ 4. Un-check box **Extend my Windows Desktop onto this Monitor**
- ☐ 5. Click **Apply**, click **OK**

Note: Connector head identification is molded on each head of supplied adapter cable as follows:

3/NAV = Navigation VGA Display (**supports analog only**) Use analog cable with supplied DVI to analog converter)

1/IMG1 = First Grayscale Display

2/IMG2 = Second Grayscale Display

Note: For best performance using Windows XP consider changing Visual Effects setting from "Best Appearance" to "Best Performance"...

Click **Start**, click **Run**, and then type **sysdm.cpl** in the **Open** box. Click the **Advanced** tab, and then under **Performance**, click **Settings**. Select **Adjust for best performance**, click **Apply**, and click **OK**